

To whom it may concern:

TECHNICAL DATA SHEET

Henry Schein Sterireel sterilization rolls

Product assortment:

REF	Description	Contents
9002993	HS-Sterireel sterilisation roll, 100mm	1 / 200 m
9002989	HS-Sterireel sterilisation roll, 100mm	1 / 100 m
9002991	HS-Sterireel sterilisation roll, 50mm	1 / 200 m
9002987	HS-Sterireel sterilisation roll, 50mm	1 / 100 m
9002990	HS-Sterireel sterilisation roll, 65 x 200mm	1 / 100 m
9002992	HS-Sterireel sterilisation roll, 75mm	1 / 200 m
9002988	HS-Sterireel sterilisation roll, 75mm	1 / 100 m
9880485	HS-Sterireel sterilisation roll, 150mm	1 / 200 m
9880486	HS-Sterireel sterilisation roll, 200 mm	1 / 200 m
9880487	HS-Sterireel sterilisation roll, 250 mm	1 / 200 m
9880488	HS-Sterireel sterilisation roll, 300 mm	1 / 200 m

Intended use:

For use as packing material for medical devices in sterilization by steam, ethylene oxide gas in healthcare establishments.


Sterilization method:

Compatible with Steam and Ethylene Oxide (EO) sterilization methods. The reels maintain sterility of the device for up to one year post sterilization under appropriate storage conditions. Recommended sealing temperature: 200°C±10°C, sealing time: 2.5 seconds.

Sterilization parameters:

Sterilization Cycle Parameter	Gravity / Steam		Ethylene Oxide Gas
Time (min)	30	4	100-120
Temperature(°C)	121	134	50
Relative humidity(%)	N/A	N/A	60-85
Sterilant Concentration	N/A	N/A	600
Drying-Aeration time	15-30 min	20-30 min	12 hrs

Process Indicator Color Change Guide:

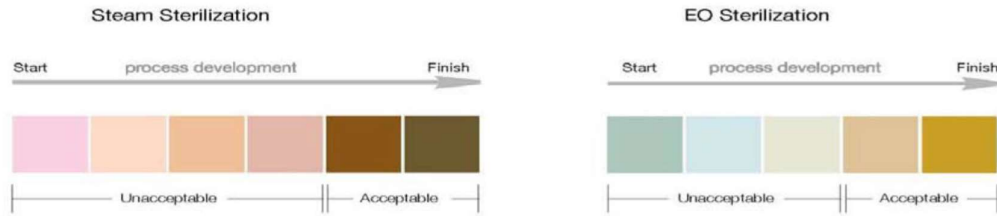

Material:

Pouches and corresponding packaging is latex-free. Non-tearing, thick multi-layer film made from a combination of polypropylene and polyethylene. High quality, porous 60 g/m² medical grade paper.

Regulatory:

In accordance with requirements of the MDD93/42/EEC. In conformity with the relevant harmonized standards EN 868-5, EN ISO 11607-1 and international Standard ISO 11140-1

Shelf-life:

Normal conditions of conservation & storage: not to be exposed to moisture and sun, must be stored at a temperature between 5°C and 40°C. Shelf life of the product: 36 months. For single-use only.



Please do not hesitate to contact us if you have any questions or comments.

 **HENRY SCHEIN INC.**
(manufacturer) **135 DURYEA ROAD**
Melville, NY 11747 USA
EC REP
HENRY SCHEIN SERVICES GmbH
MONZASTRASSE 2A
D-63225 LANGEN, GERMANY
EMAIL:CBDEUROPE@HENRYSCHEIN.COM
WWW.HENRYSCHEINBRAND.COM