EuroSept® Xtra Thermo Cleanser

(ES) Limpiador Thermo

(FR) Nettoyant thermique

(DE) HS - Flüssigreiniger Thermo

(IT) Detergente liquido termico

(NL) Thermo-reiniger

(PT) Detergente térmico

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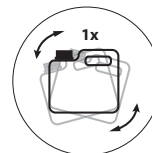
(DA) Thermo rensmiddel

(SV) Thermo rengöringsmedel

(NO) Thermo rengjøringsmiddel

Danger
Peligro
Gefahr
Pericolo
Gevaar
Perigo
Nebezpečí
Niebezpieczeństwo
Fara
Fare

**UK
CA**

UK Responsible Person
Ergo Consulting (UK) Limited
c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

UFI: XGP8-EN0D-710S-3MSR

Exclusively produced for :

 HENRY SCHEIN®

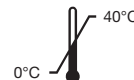
PRISMAN GmbH
Otto-Hahn-Ring 6-18
D-64653 Lorsch
Phone: +49 (0)6251-866 980-0
E-Mail: info@prisman.de

01/22 Rev.2022-09

5Le

GTIN
Data Matrix Code
LOT - HBD

EN: Type of application: Foam-free, alkaline cleaning concentrate for the automated cleaning of surgical instruments, medical-technical utensils and laboratory glass in hospitals, medical practices, and laboratories. **Instructions for use:** Shake the canister once before connecting it for the first time. Any suspended particles that may occur have no effect on the equipment or on the effectiveness. Anodised aluminium surfaces and titanium alloys must be tested for resistance as colour changes may occur. This must be taken into account, e.g. for implants with colour coding. The dosage must be carried out using a suitable dosing device and depends on the contamination level of the filling material. It is approx. 2 ml – 5 ml/l. **Recommended dosage:** For the processing of surgical instruments with subsequent thermal disinfection: Cleaning: 2 ml – 5 ml/l EuroSept@Xtra Thermo Cleanser; Neutralisation 1 ml – 2 ml/l EuroSept@Xtra Thermo Neutralizer; Rinsing: 0.3 ml – 1 ml/l EuroSept@Xtra Thermo Rinse; For the removal of blood, serum, analysis residues and nutrient medium (laboratory): Cleaning: 2 ml – 5 ml/l EuroSept@Xtra Thermo Cleanser; Neutralisation: 1 ml – 2 ml/l EuroSept@Xtra Thermo Neutralizer To prevent water stains, it is recommended to use fully demineralised water for the final rinse. Do not mix with other products! The entire dosing system should be rinsed with water before changing products. Observe the device-specific operating instructions. The reprocessing of instruments must be carried out as per the Medical Device Operator Regulation and RKI (Robert Koch Institute) guidelines using suitable procedures. The corresponding reprocessing recommendations of the instrument manufacturer must be observed as per the requirements of EN ISO 17664. EuroSept@Xtra Thermo Cleanser is not suitable for cleaning transmission instruments. **Safety instructions:** H314 Causes severe skin burns and eye damage. Wear protective gloves / protective clothing / eye protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor. Store locked up. Dispose of contents/container in accordance with all local, regional, national and international regulations. Contains potassium hydroxide, silicic acid sodium salt, sodium metasilicate. Reg. (EC) 648/2004: Contains 5-15% phosphates. All serious incidents related to the device shall be reported by the user/patient to the manufacturer and to the competent authority of the State in which the user/patient is established.

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